## DEUTSCH

**Inhalt:**

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## ENGLISH

**Contents:**

1 x scanbody
1 x laboratory screw (colour – black)
1 x implant screw (colour – titanium)

**Indication:**

The scanbody is used to record the position of the implant in the mouth, or on the model during the scanning process. Selection must be carried out according to the compatibility of the implant system and the platform diameter. Both the position of the implant and the rotation are depicted due to the asymmetric form of the scanbody. It is important to select the correct implant library for referencing the position.

**Contraindication:**

It is imperative to ensure that only components are only combined with compatible implant systems. This also applies to the corresponding platform diameter.

- The DD scanbody may not be used in the following situations:
  - Intolerance with regard to the components contained
  - If the implant position displays angulation of more than 18°
  - The DD scanbody may only be used by expert personnel in combination with the current applicable instructions for use.
  - The DD scanbody may only be use in combination with the DD implant library.
  - If the implant interface is damaged, the scanbody may not be used.

**Material:**

The scanbody base and implant screw are made of Ti6Al4V (medical grade 5) ASTM 136 ELI. The scanbody super-structure is made of PEEK (Polyether ether ketone). The lab screw is made of stainless steel DIN 1.4305.

**Instructions for use:**

The scanbody is supplied for use on the model with a laboratory screw (black) and with a titanium-coloured implant screw for use in the patient’s mouth. A maximum torque of 10 Ncm should be applied. For use in the model, only use the black laboratory screw.

**Cleaning, disinfecting and sterilisation:**

The scanbody and implant screws are delivered to the dental laboratory or dental practice in a generally clean, but not sterile, condition, in appropriate secure packaging. After processing and before use with patients, the components must be cleaned, disinfected and sterilised using the usual process.

**Cleaning/disinfection:**

- Automatic cleaning, for example, with the Vario TD programme from Miele or the following manual cleaning procedure:
  - Rinse with cold tap water for 1 minute.
  - Rinse the screw channel using a disposable syringe (min. 10 ml)
  - Clean at 60° C +/- 2° C with a neodisher cleaner, e.g. Septo MED or LM 2 from Dr. Weigert for 10 minutes in concentration of 5% in an ultrasound unit
  - Neutralisation with cold distilled water for 2 minutes.
  - Rinse with distilled water 60° C for 10 minutes in an ultrasound unit

**Sterilisation:**

Double sealing of products in sterilization wrap according to EN ISO 11607-1.
Steam sterilization in a suitable sterilizer unit with 3x fractional pre-vacuum: 134° C (273° F) for 5 minutes, drying time: 20 minutes. Then store in sterile packaging.

**Possible side effects:**

Allergies or skin irritation due to use of alloys are possible in very rare cases.

**Interactions:**

Different alloys in the mouth cavity can lead to a galvanic reaction in the event of direct contact.

**Warning:**

The scanbody may not show any signs of damage and should be checked before the scanning process. Only flawless, correct insertion of the scanbody guarantees a faultless scanning process and precise representation of the implant position.

**Warranty:**

Dental Direct guarantees the abutments for 10 years, based on expert processing and where the processing instructions have been followed. Our products are subject to strict quality controls and correspond to the latest standards of technology.

*Our products are subject to continual development, so we reserve the right to make changes. You will find the latest version of the instructions for use on our homepage at: [www.dentaldirekt.de](http://www.dentaldirekt.de). This version replaces all previous version.*

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